



hygie™

C A N A D A

Classic Hygienic covers tutorial

Oxo-Bio



Set-up

IMPORTANT: Before using this apparatus, it is suggested that you wear gloves. ❶ Remove the super absorbent pad from the hygienic cover. Set aside. Place your hand inside a corner of the cover and hold the support **hy₂₁** (thumb on top). ❷ Pull the hygienic cover towards the back and over the support, to wrap it. ❸ Turn the support towards the floor and secure the hygienic cover by pulling on the drawstrings to completely cover the support. ❹ Flatten the hygienic cover inside the support's cavity. Place the absorbent pad at the bottom of the support, in its cavity. The support **hy₂₁** is ready for use.



Remove



❺ While holding the support **hy₂₁** evenly, remove the hygienic cover with the help of the drawstrings. ❻ Hold hygienic cover at its base and pull on drawstrings to remove air from the hygienic cover. ❼ Twist hygienic cover at least 3 times before securing it with a knot. ❽ Tie a solid knot at the base of the hygienic cover and DISPOSE. **IMPORTANT: After disposal of the hygienic cover, remove the gloves and wash your hands.**



Placement of the Support **hy₂₁**

PATIENT WITH MINIMAL ASSIST

NOTE: The support **hy₂₁ must be wrapped with a hygienic cover, and the super absorbent pad should be placed at the bottom of the support **hy₂₁** before it is placed under the patient.** ❶ Have the patient bend and prop up the knees, and lift their pelvis. ❷ Hold the support **hy₂₁** by the built-in handle, found opposite the side with the extended edge. ❸ Make sure the patient's "glutes" cover a third (1/3) of the support **hy₂₁** so they are vis-à-vis the lateral handles found at the bottom of the support **hy₂₁**. ❹ Have the patient lower their pelvis onto the support **hy₂₁**, which is ready for use.

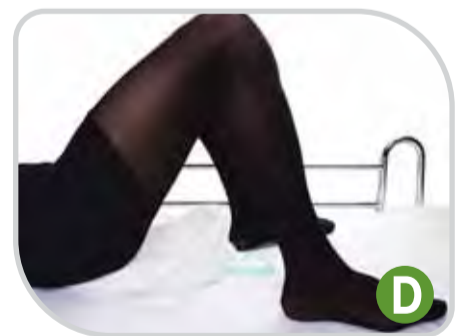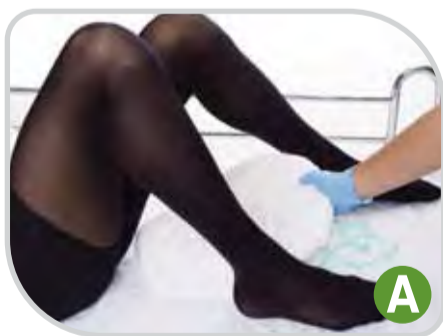

DEPENDENT PATIENT

NOTE: The support **hy₂₁ must be wrapped with a hygienic cover, and the super absorbent pad should be placed at the bottom of the support **hy₂₁** before it is placed under the patient.** Turn patient on side. Hold the support **hy₂₁** by one of the built-in lateral handles, found opposite the side with the extended edge. Place the support **hy₂₁** vis-à-vis the patient's pelvis, and turn patient on back to position over the support **hy₂₁**. Make sure the patient's gluteus cover a third (1/3) of the support **hy₂₁**, which is ready for use. **REMOVAL OF THE SUPPORT **hy₂₁**:** After patient use of the support **hy₂₁**, it is suggested to wait until the gelling process of the liquid matter has started before proceeding with removal of the support **hy₂₁**.

