

Classic Hygienic covers tutorial

Oxo-Bio



Set-up **IMPORTANT:** Before using this apparatus, it is suggested that you wear gloves. ❶ Remove the super absorbent pad from the hygienic cover. Set aside. Place your hand inside a corner of the cover and hold the bedpan (thumb on top). ❷ Wrap the hygienic cover around the bedpan. ❸ Turn the bedpan so it faces the floor and secure the hygienic cover by pulling on the drawstrings to cover the bedpan completely. ❹ Flatten the hygienic cover inside the bedpan's opening. Place the absorbent pad in the bedpan. The bedpan is ready for use.



Remove ❺ While maintaining the bedpan even, remove the hygienic cover with the help of the drawstrings. ❻ Hold hygienic cover at its base and pull on drawstrings to remove air from the hygienic cover. ❼ Twist hygienic cover at least 3 times before securing it with a knot. ❽ Tie a solid knot at the base of the hygienic cover and DISPOSE. **IMPORTANT:** After disposal of the hygienic cover, remove the gloves and wash your hands.



Installation and removal

of the Care Bag Classic hygienic cover on the pail or seat of the commode chair



SET-UP: **A** Remove the super absorbent pad from the hygienic cover, and set aside. **B** Lift the lid and the seat of the commode chair. **C** Insert the bottom of the hygienic cover through the opening of the seat. **HYGIENIC COVER ON PAIL:** **D** Turn the edges of the hygienic cover over the brim of the pail. **E** Put pail in place. **F** Place the super absorbent pad into the hygienic cover at the bottom of the pail, and lower the seat. **HYGIENIC COVER ON SEAT:** **D** Turn the edges of the hygienic cover over the seat of the commode. Make sure that the edge of the hygienic cover is folded under the seat. **E** Lower seat of the commode. Make sure the pail is present so it may offer the hygienic cover some support. Verify that the bottom of the hygienic cover is taut and is resting against the pail. **F** Lower the seat and place the super absorbent pad into the hygienic cover. **G** The commode chair is ready to be used.



REMOVAL (see images 6, 7 and 8): **A** After the commode chair has been used, to make the most of the hygienic cover's assets, it must be removed with care and closed securely. Using the hygienic cover eases the maintenance of the commode chair and helps limit the spread of bacteria and will lock in odours. **B** To remove the Care Bag hygienic cover and to close it securely, carefully bring its sides up, towards the opening of the seat and over the pail. **C** Remove with care. **D** Take hold of the hygienic cover at its base, and pull on the ties to tighten the opening and evacuate the air from it. **E** Twist the hygienic cover several times until a rope-like effect is obtained. **F** Tie a solid flat knot at its base to tie it shut and to secure its contents. DISPOSE.